

Vacuum Plate Package Contents:

- Vacuum Plate (LLV-01043) with hose barb installed
- Two Proximal Sealing Gaskets
- 2-foot length of tubing for use as an external hose (1/4" OD, 1/8" ID)
- Four M6 x 20 Flat Capscrews
- Four M6 x 25 Flat Capscrews

Vacuum Pyramid Package Contents:

- Vacuum Pyramid (LLV-01044) with hose barb installed
- 2-foot length of tubing for use as an external hose (1/4" OD, 1/8" ID)
- Two Proximal Sealing Gaskets



IMPORTANT:

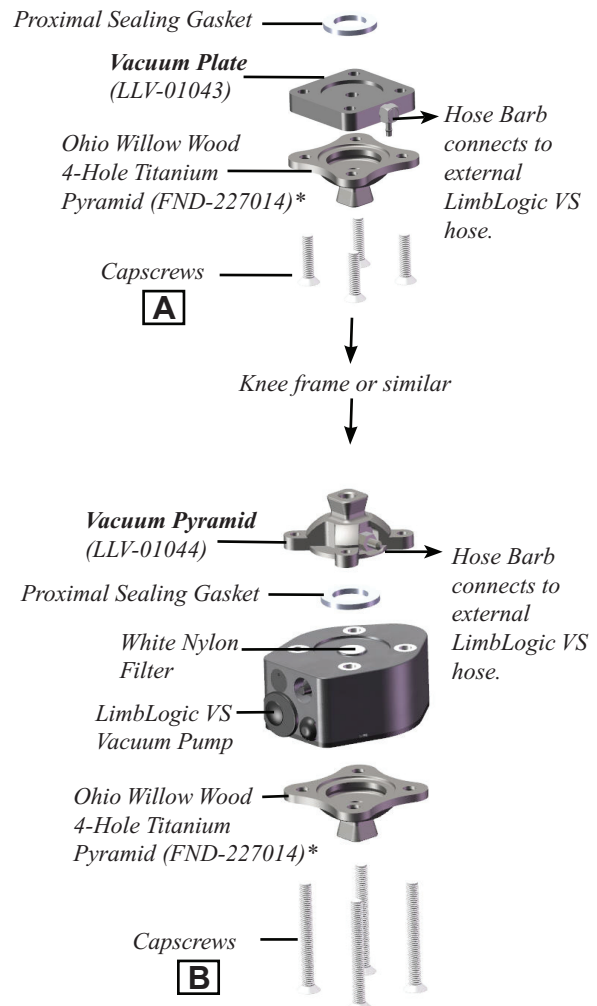
Use the Vacuum Plate and Vacuum Pyramid ONLY in the configurations shown.

Attach the desired components using flat capscrews of the appropriate length, as indicated by the chart on the back of this document (refer to Column A for the Vacuum Plate, and Column B for the Vacuum Pyramid). Apply Loctite 242 Removable Threadlocker (or equivalent) and tighten the screws to 9 ft-lbs (12 Nm).

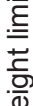
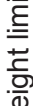
Notes:

- **Contact Ohio Willow Wood if you need a bolt length that is not included in the kit.**
- **At least 3/16" (4.7mm) of the capscrews must be engaged. Because the threads inside each adapter do not necessarily extend all the way to the surface of the adapter, be sure to measure the amount of thread engagement with the actual threaded portion of the adapter.**

The hose barb connects the LimbLogic VS to the external hose. Attach one end of the external hose to the Vacuum Pump and the other end to the socket as desired.



*The following Ohio Willow Wood adapters may also be used in this configuration:

		Capscrew length when attaching to the following:	
		A Vacuum Plate (LLV-01043)	B Vacuum Pyramid (LLV-01044)
	Component		
 Weight limit: 250 lbs/115 kg	FND-134130 Aluminum 4-Hole 30mm Adapter	20mm	50 or 55mm
	FND-227014 4-Hole Titanium Pyramid Adapter	20mm	50 or 55mm
	FND-227004 4-Hole Titanium Pyramid Receiver	20mm	50 or 55mm
	FND-224120 Titanium 4-Hole Pyramid Adapter, 20mm Offset	20mm	50 or 55mm
	FND-134125 Slide Adapter, Aluminum, 4-Hole Pyramid	25mm	50 or 55mm
	FND-134150 Slide Adapter, Aluminum, Rotating, 4-Hole Pyramid	25mm	50 or 55mm
	PFD-0114 Pathfinder® II Through 4-Hole Adapter with Valve	20mm	50 or 55mm
	FND-228002 Titanium Rotatable Pyramid Receiver	25mm	55mm
FND-228003 Titanium Rotatable Pyramid Adapter	25mm	55mm	
 Weight limit: 350 lbs/160 kg	MHD-224125 Magnum Slide Adapter, Titanium, 4-Hole Pyramid	25mm	50 or 55mm
	MHD-224150 Magnum Slide Adapter, Titanium, 4-Hole Pyramid	25mm	50 or 55mm
	MHD-PR-4 Magnum Titanium 4-Hole Pyramid Receiver	20mm	50 or 55mm
	MHD-PA-4 Magnum Titanium 4-Hole Pyramid Adapter	20mm	50 or 55mm

Warranty

The warranty for the **Vacuum Plate** and **Vacuum Pyramid** is one year (allowing three weeks from the date of purchase for installation). Use of the **Vacuum Plate** or **Vacuum Pyramid** for amputees whose modified body weight is more than 350 lbs (160 kg) or who engage in extremely high and abusive activity is against Ohio Willow Wood Company’s recommendations and will void the one-year warranty. Modified body weight is defined as the weight of the amputee plus any loads carried by the amputee. “Extremely high and abusive activities” are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual’s natural limbs; and activities that expose the prosthesis to corrosives such as salt water.

Warranty Disclaimer

Ohio Willow Wood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. **OHIO WILLOW WOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.** This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. Ohio Willow Wood’s liability shall not exceed the purchase price of the product. **Ohio Willow Wood shall not be liable for any indirect, incidental, or consequential damage.**

Ohio Willow Wood Retention of Rights

Ohio Willow Wood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.