

Joint Solutions

Earthwalk™ 2 Foot and Ankle Kit

The Earthwalk 2 Flexible Keel Foot and Ankle System provides freedom of mobility that is more natural than any other flexible keel foot and ankle system on the market today. The revolutionary Earthwalk 2 Ankle provides an exceptionally smooth motion and more stability. The “late stop” ankle feature allows for easy roll over so the foot can maintain foot flat longer. This action is important for less-active amputees who require more stability and less dynamic response. The Earthwalk 2 Foot and Ankle Kit is lighter than many other foot/ankle combinations on the market and the new, durable design makes adjustments to the foot easier since the ankle is sleeker and taller. Offered in four different stiffnesses, the ankle is virtually maintenance-free—its parts never need to be replaced. Packaged with the Earthwalk 2 Flexible Keel Foot, this unit provides lower activity amputees (U.S. Levels 1 and 2) with outstanding smoothness at a value-minded price. The ability to maneuver a variety of terrains is important to many amputees, and with the Earthwalk 2 this is once again possible.



- **U.S. Activity Levels 1 & 2**
- **Flexible Keel Foot/Ankle Combination**
- **4 Toe Resistances**
- **4 Ankle Stiffnesses**
- **3 Interchangeable Heel Inserts Included**

SPECIFICATIONS

WEIGHT	643 g*
HEEL INSERTS INCLUDED	Soft=12 g; Medium=15 g; Firm=20 g
PATIENT WEIGHT LIMIT	250 lbs/115 kg**
U.S. L-CODE SUGGESTION	5986, 5972
HEEL HEIGHT	Women - 3/8" (10 mm) or 1 1/8" (30 mm) Men - 3/4" (18 mm) or 3/8" (10 mm)
AVAILABLE SIZES	Women - 22 to 27 cm, Men - 23 to 31 cm
TOE RESISTANCE	Very Low, Low, Regular, High
COSMESIS	Life-like Buff, Tan or Medium Brown
TRIAL PERIOD	If the amputee rejects the Earthwalk™ 2 within 30 days, the Earthwalk™ 2 may be returned to Ohio Willow Wood for full credit.
U.S. WARRANTY	One year and three weeks from purchase

* Based on size 26 cm foot without heel insert.

** Body weight plus any loads normally and routinely carried cannot exceed this weight limit.

GENERAL GUIDELINES

Use the amputee’s foot size and Adjusted Body Weight to determine the recommended toe resistance from the charts. “Adjusted Body Weight” is the amputee’s body weight plus any loads normally or routinely carried.

Note 1: To provide more of a toe lever for the amputee, select a foot with the next higher level of toe resistance on the chart, but do not change the ankle selection.

Note 2: Amputees who plan to use their prostheses for running should select a Pathfinder® II or a Magnum Foot, or similar high performance foot in order to obtain the appropriate level of response.

Joint Solutions

GENERAL GUIDELINES

TOE STIFFNESS

Very Low Toe	Low Toe	Regular Toe	High Toe
Less than 150 lbs Less than 70 kg	151-175 lbs 71 - 80 kg	176 - 220 lbs 81-100 kg	221 - 250 lbs 101 - 115 kg

ANKLE STIFFNESS

Amputee Weight	Ankle Selection	Part No.
Less than 150 lbs (70 kg)	RED	EWA2-R
151-175 lbs (71-80 kg)	BLUE	EWA2-B
176-220 lbs (81-100 kg)	YELLOW	EWA2-Y
221-250 lbs (101-115 kg)	GREEN	EWA2-G

ORDERING GUIDELINES: Earthwalk™ 2 Kit

	EWF2W-26LM-3H-Y	Earthwalk™ 2 Kit SAMPLE Part Number
Name Code	EWF2W	EWF2M = Earthwalk™ 2 Kit - Mens EWF2W = Earthwalk™ 2 Kit - Womens
Size of Foot	26	Mens = 23 to 31 cm Womens = 22 to 27 cm
Side of Foot	L	L = Left R = Right
Color	M	B = Buff M = Medium Brown T = Tan
Toe Stiffness	3	1 = Very Low 2 = Low 3 = Regular 4 = High
Heel Height	H	L = Low 10 mm (3/8") M = Medium 18 mm (3/4") Mens only H = High 30 mm (1 1/8") Womens only
Ankle Stiffness	Y	R = Red B = Blue Y = Yellow G = Green

